COVIDIEN™

ForceTriad™

Bipolar Resection Cords

REF FT0021S

REF FT0022W

en Instructions for Use

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sv Bruksanvisning

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ForceTriad™

REF FT0021S **Bipolar Resection Cord, Storz, Reusable**
15' (4.6 m)

REF FT0022W **Bipolar Resection Cord, Wolf, Reusable**
15' (4.6 m)

For use with the ForceTriad energy platform with software version 3.30 or greater.

Footswitching

Intended Use

Cords are to electrically connect bipolar-resection electrosurgical instruments to the ForceTriad energy platform with software version 3.30 or greater with the Bipolar Resection feature activated. The cords have a Smart™ connector that connects to the LigaSure 2/Bipolar Resection receptacle.

The Bipolar Resection Cord is intended to be used for the following procedures in a saline environment: Transurethral Resection of the Prostate (TURP), Transurethral Resection of the Bladder (TURB), and hysteroscopic resection.

The system will perform within specifications with the following Wolf™* and Storz™* electrodes and working elements:

Wolf Medical Instruments	Catalog Number
Fr Cutting Loop 0.3mm round wire	4622.1313 22
Working element for uro resection	8680.224
Working element for hysteroscopic resection	8653.224

Storz Surgical Instruments	Catalog Number
Cutting by means of a finger grip, rest position = electrode outside sheath	27040 DB
Cutting by means of a spring, rest position = electrode inside sheath (List continues on next page)	27040 EB

Storz Surgical Instruments (continued)	Catalog Number
---	----------------

Cutting Loop, bipolar	27040 GPV
Cutting Loop, bipolar, diameter 0.4 mm	27040 GPV40
Cutting Loop, bipolar	27040 GP
Cutting Loop, bipolar, diameter 0.4 mm	27040 GP40
Cutting Loop, bipolar	26040 GP

Warning

This product is non-sterile. Clean and sterilize cords before the first and every use.

DO NOT sterilize the cord using low temperature gas plasma, low temperature peracetic acid, dry heat, soaking, or irradiation. Failure to comply may result in patient injury and/or reduced product performance.

Cleaning

The cleaning efficacy has been validated for using either of the manual or mechanical cleaning methods described in the following sections.

Caution

Use only recommended cleaning solutions. Do not use solvents, lubricants, or other chemicals to clean the reusable cord. Some disinfection solutions are corrosive to the connectors.

Manual Cleaning

- Soak cords in an enzymatic cleaner, such as Klenzyme® in accordance with the manufacturer's recommendations
- Remove gross particulate with a soft-bristle brush or cloth
- Clean the cord's instrument receptacle by inserting a small, soft-bristle brush to remove particulate
- Rinse and dry with a lint-free cloth

Mechanical Cleaning

Clean in a low impingement, Steris model Reliance® 444 single chamber washer/disinfector.

1. Pre-rinse in cold water for 30 seconds.
2. Wash with H-Klenz® II (sodium metasilicate, high alkaline), 4 minutes, 140° F (60° C)
3. Rinse with C-A Klenz® (neutralization cycle, citric acid), 2 minutes, 150° F (66° C)
4. Rinse with water, 15 seconds, 140° F (60° C)
5. Dry, 15 minutes, 180° F (82° C)
6. Cool before autoclave cycle, (15 min. - 30 min.)

Sterilization

The sterility of the cord has been validated for the following parameters:

1. Wrap each reusable cord separately in a double layer of central-supply-room (CSR) sterilization wrap.

Notice

Separating reusable cords during sterilization maximizes the life of the cord. Heat from sterilization softens the cord insulation. Contact with other cords or instruments during sterilization can damage the reusable cord.

2. Use the following guidelines for sterilization.

Steam Sterilization - Wrapped

Temperature	Type	Sterilization Time
270° F to 275° F (132° to 135° C)	Prevac	3 min.
250° F (121° C)	Prevac	15 min.
270° F to 275° F (132° to 135° C)	Gravity	10 min.
250° F (121° C)	Gravity	30 min.

Temperature	Type	Sterilization Time
For specific dry times, refer to the documentation from the manufacturer of the sterilization system in use and hospital guidelines.		

Steam Sterilization - Unwrapped (Flash)

Temperature	Type	Sterilization Time
270° F to 275° F (132° to 135° C)	Prevac	4 min.
270° F to 275° F (132° to 135° C)	Gravity	10 min.

For specific dry times, refer to the documentation from the manufacturer of the sterilization system in use and hospital guidelines.

These procedures have been validated by Covidien as capable of preparing the FT0021S and FT0022W Bipolar Resection cords for re-use. It is the responsibility of the processor to ensure that sterilization is performed using equipment, materials, and personnel that will achieve the desired results. This requires validation and routine monitoring of the

process. Any deviation by the processor from the instructions provided should be properly evaluated for effectiveness or potential adverse consequences.

Product Life Expectancy

The product life has been validated for a 50-use cycle using the following cleaning and sterilization processes:

- Manual cleaning followed by vacuum-assisted (Prevac) steam sterilization at 280° F (138° C) for 30 minutes, and a 45-minute dry time
- Mechanical cleaning followed by vacuum-assisted (Prevac) steam sterilization at 280° F (138° C) for 30 minutes, and a 45-minute dry time

Assembly

Warning

Malfunction may be caused by a loose or incorrect connection of the cord to the generator or instrument. Worn cords cannot be repaired. Replace cords on a routine basis to maintain optimum function and reliability in operating room use.

Warning

Connect accessories to the proper receptacle. Improper connection of accessories may result in potentially hazardous conditions.

Inspect the cord insulation before and after each use for cracks, nicks, cuts, dents, or depressions which may decrease the insulation effectiveness. If the cord is compromised, do not use. Unintended thermal injury could occur.

Caution

Always disconnect the cord from the instrument or generator by gripping the connector. Do not pull on the cord when disconnecting from the instrument and generator. Improper disconnection may damage the cord insulation.

Prior to use, the reusable cord and instrument should be cleaned and sterilized.

1. Inspect the cord for the following end-of-life indicators:
 - Cracks, nicks, cuts, dents or depressions in the cord insulation and connectors

- Exposed wires
- Bent or loose plugs
- Loose receptacles in connectors

If any of these indicators are identified, replace the damaged cord with a sterilized cord in good condition.

2. Connect the reusable cord to a bipolar-resection instrument specified in the *Reference Documentation* section on page 7. Ensure that the cord receptacles are fully seated against the instrument's connectors.
3. Connect the reusable cord Smart connector with the barcode facing up to the LigaSure 2/Bipolar Resection receptacle on the energy platform.

The cord is now ready to be used with a ForceTriad energy platform with the Bipolar Resection feature activated.

Reference Documentation

For more information about using the Bipolar Resection feature, refer to these documents:

- *ForceTriad Energy Platform User's Guide* (v. 3.30 or greater)
- *ForceTriad Bipolar Resection Footswitch (FT6009) Instructions for Use*

Troubleshooting

For additional questions, contact a technical service representative:

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Consult
instructions
for use

Part No. 1024666

COVIDIEN, COVIDIEN with logo and the Covidien logo are U.S. and/or internationally registered trademarks of Covidien AG.

Klenzyme, H-Klenz, C-A Klenz, and Reliance are registered trademarks of the STERIS Corp.

^{TM&C} are trademarks of their respective owners.

All other brands are trademarks of a Covidien company.

© 2009 Covidien.

 Manufactured for Valleylab, a division of Tyco Healthcare Group LP, Boulder, CO 80301-3299 USA.

 Tyco Healthcare UK Ltd., Gosport, PO13 0AS, UK.

Made in China. Printed in USA.

www.covidien.com

REV 12/2009