

8 steps to start up the iCare+ column

1 Switch on the column

The green switch lights



2 Check that iCare starts

The logo flashes rapidly : otherwise, check that the rear switch is on «I» position



3 Wait 90 seconds for iCare to start

The logo flashes slowly





4 Check that the camera and the light cable are connected

Connect the camera on the right and the light cable on the left



5 Check the proper functioning of the monitor

The message «Light source in Std-by mode» must appear on the screen

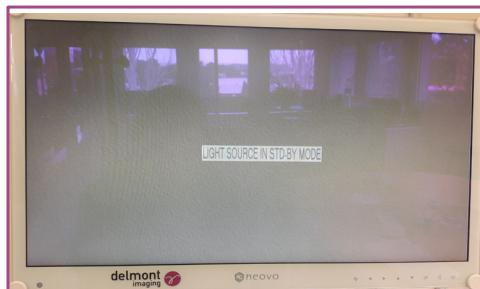
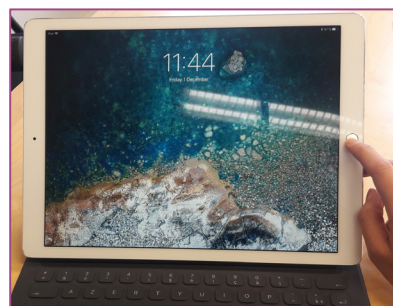


6 Turn on the iPad

Press on the main button.


7 Check that the iPad is connected to the iCare Wifi

Go to the Wifi tab of the iPad settings

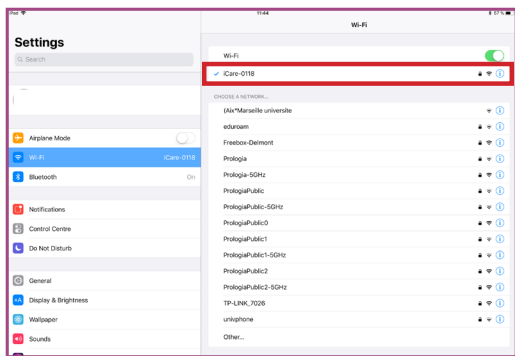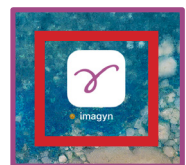

8 Launch imagyn

How to perform an exam with iCare+


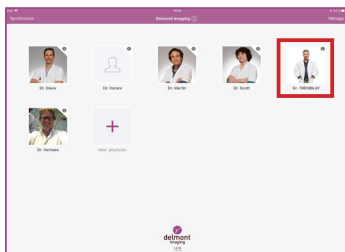


1 The doctor selects her/his account
She/he must enter her/his private code

The doctor creates a new patient

(or selects the patient in the worklist if already existing)

2

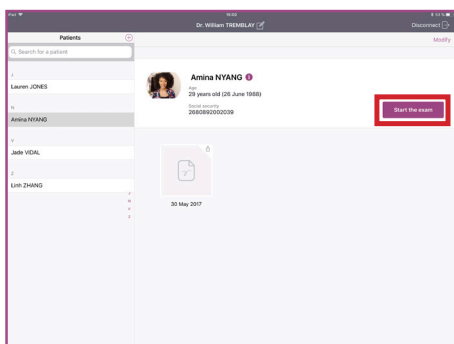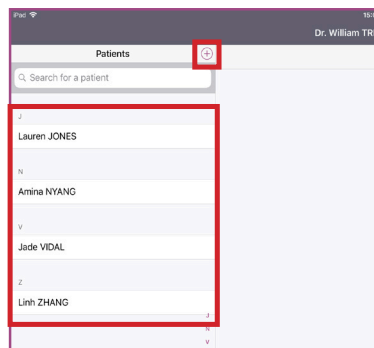

3 Start the exam

Some fields of the report must be completed (option). After validation, the image of the camera also appears on the iPad.



4 Activate the light source
Press on the button on the right side of the iCare CCU

5 Carry out a white balance

- Film a white surface
- Press the AWB button until «AWB in progress...» appears
- Keep on filming a white surface until «AWB OK» appears



Between two hysteroscopies, don't switch off the column or iCare, just put the light source in standby mode.