

Wired connection iCare - iPad

With the elements of this kit, change your WiFi connection to wired for iCare and imagyn functioning.

Supplied items:

- One iPad adapter,
- One Ethernet cable,
- One USB 3.0 - Ethernet adapter.



Connections:

1 Connect the Ethernet cable to the iPad adapter.



2 Connect the second end of the Ethernet cable to the Ethernet adapter.





3 Connect the iPad adapter.



4 Remove one of the two WiFi keys at the back of iCare and plug the Ethernet adapter.



5 In the iPad settings, view the appearance of the «Ethernet» tab to check the well functioning of this configuration.

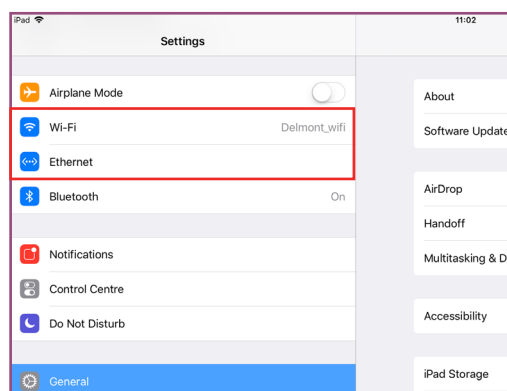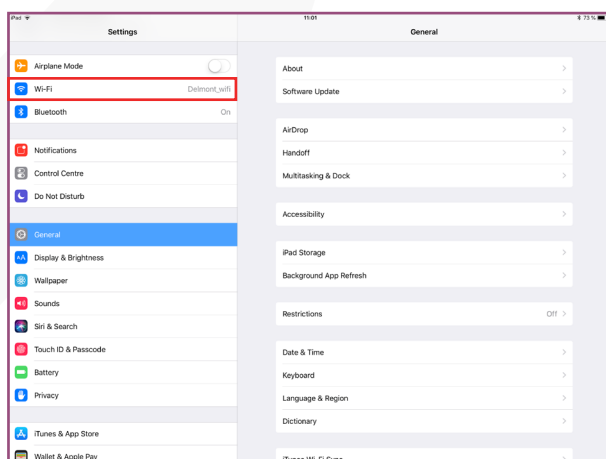

- 6** To set iCare in wired configuration instead of WiFi, press and hold the right button on the camera head 5 times while iCare is in standby. To switch back to WiFi configuration, press 5 long presses in off standby mode.



- 7** At each configuration change, restart iCare to finalize the status change.

Note:



You can charge your iPad even with this wired connection, using the supplied iPad adapter plug [1]. A USB output [2] is also available to transfer files from a USB stick to the iPad (only in this way, image captures made during the exam do not save to this stick but only to the storage key on the back of the iCare).

delmont imaging

Athélia V - 390, Av. du Mistral - 13600 La Ciotat

Tel. : +33(0) 9 51 51 30 30 - fax : +33(0) 9 57 51 31 00

www.delmont-imaging.com - contact@delmont-imaging.com

SAS au capital de 237 200 euros - RCS Marseille 817 710 411 - N° de TVA Intracommunautaire : FR 81 817 710 411