Compatible reprocessing solution for endoscopes

REF D300 100 000 to D300 100 200

Manufacturer	Solution	Step	
Alconox	Tergazyme	Cleaning	✓
Alkapharm	Alka 100	Disinfection	✗
Anios	Anioxide 1000	Disinfection	✓
Anios	Actanios HLD	Disinfection	✗
Anios	Actanios LDI	Cleaning	✗
Anios	excel D	Disinfection	✗
Anios	Hexanios G+R	Disinfection	✗
Anios	Steranios 2%	Disinfection	✗
Antiseptica	Kombi Instru N	Disinfection	✓
Antiseptica	Triacid N	Disinfection	✓
Bode	Bodedex forte	Cleaning	✓
Bode	Korsolex Extra	Disinfection	✓
Bode	Korsolex plus	Disinfection	✓
Certol	ProEZ foam	Cleaning	✓
Dekomed	Deko Neutral Detergent	Cleaning	✓
Dekomed	Deko Rinse	Cleaning	✓
Dr. Schuhmacher	Cleanisept Wipes	Disinfection	✗
Dr. Weigert	Neodisher endo DIS active	Disinfection	✓
Dr. Weigert	Neodisher Endoclean	Cleaning	✓
Dr. Weigert	Neodisher Endosept GA	Disinfection	✓
Dr. Weigert	Neodisher Mediclean	Cleaning	✓
Dr. Weigert	Neodisher Mediclean Forte	Cleaning	✓
Dr. Weigert	Neodisher Mediklar	Cleaning/ Disinfection	✓
Dr. Weigert	Neodisher Septo 3000	Disinfection	✓
Dr. Weigert	Neodisher Septo Active	Cleaning/ Disinfection	✓
Dr. Weigert	Neodisher Septo DA	Disinfection	✓
Dr. Weigert	Neodisher Septo DN	Disinfection	✓
Dr. Weigert	Neodisher Septoclean	Disinfection	✓
Dr. Weigert	Neodisher Xtreme Power	Cleaning	✓
Ecolab	Actichlor	Disinfection	✗
Ecolab	Sekusept aktiv	Disinfection	✓
Ecolab	Sekusept Multienzyme	Disinfection	✓
Ecolab	Sekusept plus	Disinfection	✗
Ecolab	Sekuseptforte S	Disinfection	✓
Germitec	Antigermix	Disinfection	✗
Getinge	Poka-Yoke DLC	Cleaning	✓
HMI	HMI Peroster CE	Disinfection	✗
IMS Italy	Adaspor	Disinfection	✓
Johnson & Johnson	Enzol	Cleaning	✓
Johnson & Johnson	Evotech	Washer/ Disinfector	✓
Johnson & Johnson	Cidex/ Cidex oPA	Disinfection	✓
Johnson & Johnson	Cidex OPA-C	Disinfection	✓
Johnson & Johnson	Cidezyme	Cleaning	✓
Johnson & Johnson	Sterad	Sterilization	✓
Lancer UK	Aperlan Fibrocleaner	Disinfection	✓
Lancer UK	Aperlan P-Y A-B	Disinfection	✓
Lancer UK	Lancerlcean	Cleaning	✓
Lancer UK	Neutral Detergent	Cleaning	✓
Lanxess	Rely+On Perasafe	Disinfection	✓
Metrex	oPA plus	Disinfection	✓
Micro- Scientific	Micro-Cide 28	Disinfection	✓
Olympic chemical	Sprayclean 2000	Cleaning	✓
Olympus	Cleaner	Cleaning	✓

Compatible reprocessing solution for endoscopes

REF D300 100 000 to D300 100 200

Manufacturer	Solution	Step	
Olympus	Disinfectand	Disinfection	✓
PFEmedical	Triosafe	Disinfection	✗
Pliwa	Trioton Rapid AF	Disinfection	✗
REDA	REDA Multi-zyme	Disinfection	✗
Renosem	Reno-S30	Sterilization	✗
Ruhof	Endozime Xtreme Power	Cleaning	✓
Saraya/ Olympus	Aceceide	Disinfection	✓
Schülke	Gigasept AF forte	Disinfection	✓
Schülke	Gigasept FF (neu)	Disinfection	✓
Schülke	Gigasept Instru AF	Disinfection	✓
Schülke	Mikrozid AF liquid	Disinfection	✓
Schülke	Thermosept DK	Disinfection	✓
Schülke	Thermosept ED	Disinfection	✓
Schülke	Thermosept ER	Disinfection	✓
Schülke	THP protect	Disinfection	✓
Sealed Air	Oxivir Wipes	Cleaning	✗
Soluscope	Soluscope P/ A/ C +	Washer/ Disinfector	✓
Steelco	EW2	Cleaning/ Disinfection	✓
Sterilox	Safer plus	Disinfection	✗
Steris	Hamo 100	Cleaning	✓
Steris	Prolystica® 2X concentrate	Cleaning	✓
Steris	Resert	Disinfection	✗
Steris	Revital-ox	Cleaning	✓
Steris	System 1E	Sterilization	✓
Steris	V-Pro	Sterilization	✓
Tristel	Perastel Multi Shot	Disinfection	✓
Tristel	Trio 25	Disinfection	✓
Tristel	Tristel Wipes	Disinfection	✓
Wassenburg	EndoHigh Detergent	Disinfection	✓
Wassenburg	EndoHigh PAA	Disinfection	✓

www.delmont-imaging.com/downloads

EN-Read the manual carefully before use. A physical copy can be supplied on request.

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