

Caution with reusable electrodes

REF D300 140 000 to D300 140 200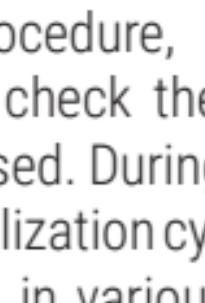


Before each procedure, it is the user's responsibility to check the integrity of the electrode to be used. During successive use, washing and sterilization cycles, an electrode can be damaged in various ways, such as:


1. Alteration of the electrode cutting wire (bending and/or deformation)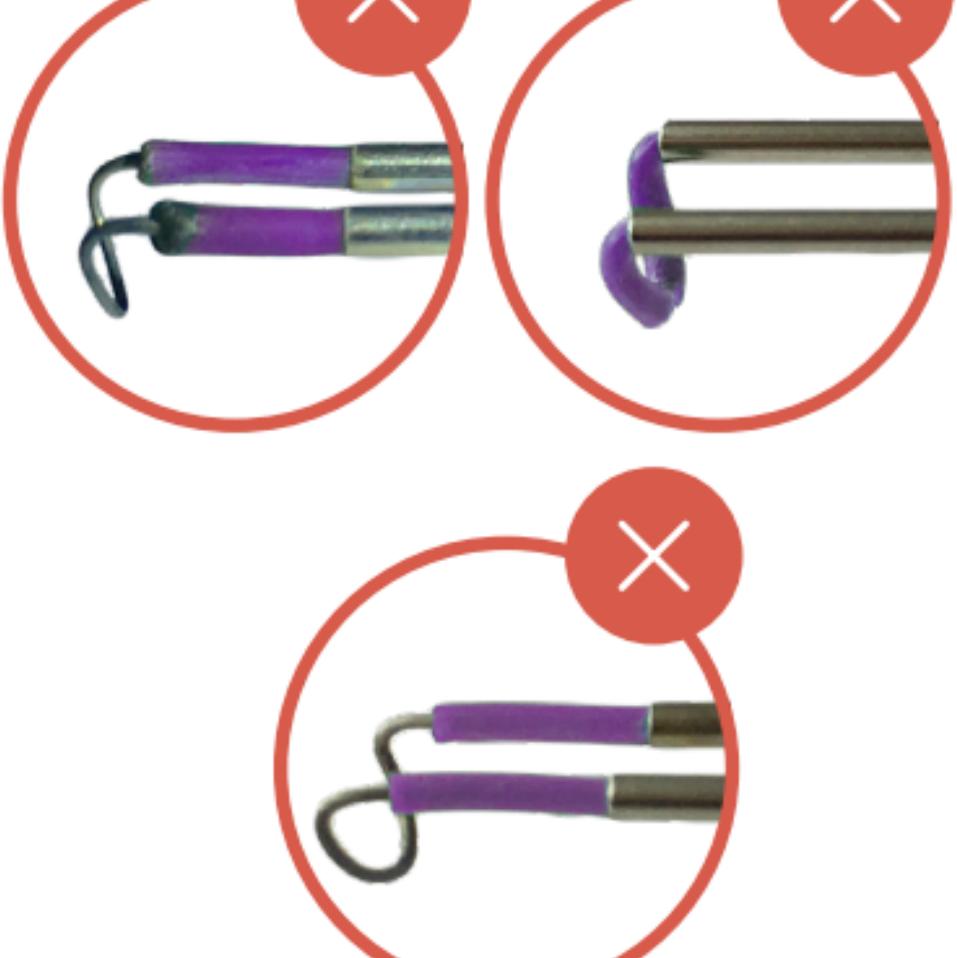

2. Alteration of insulators (burns, lacerations, displacement)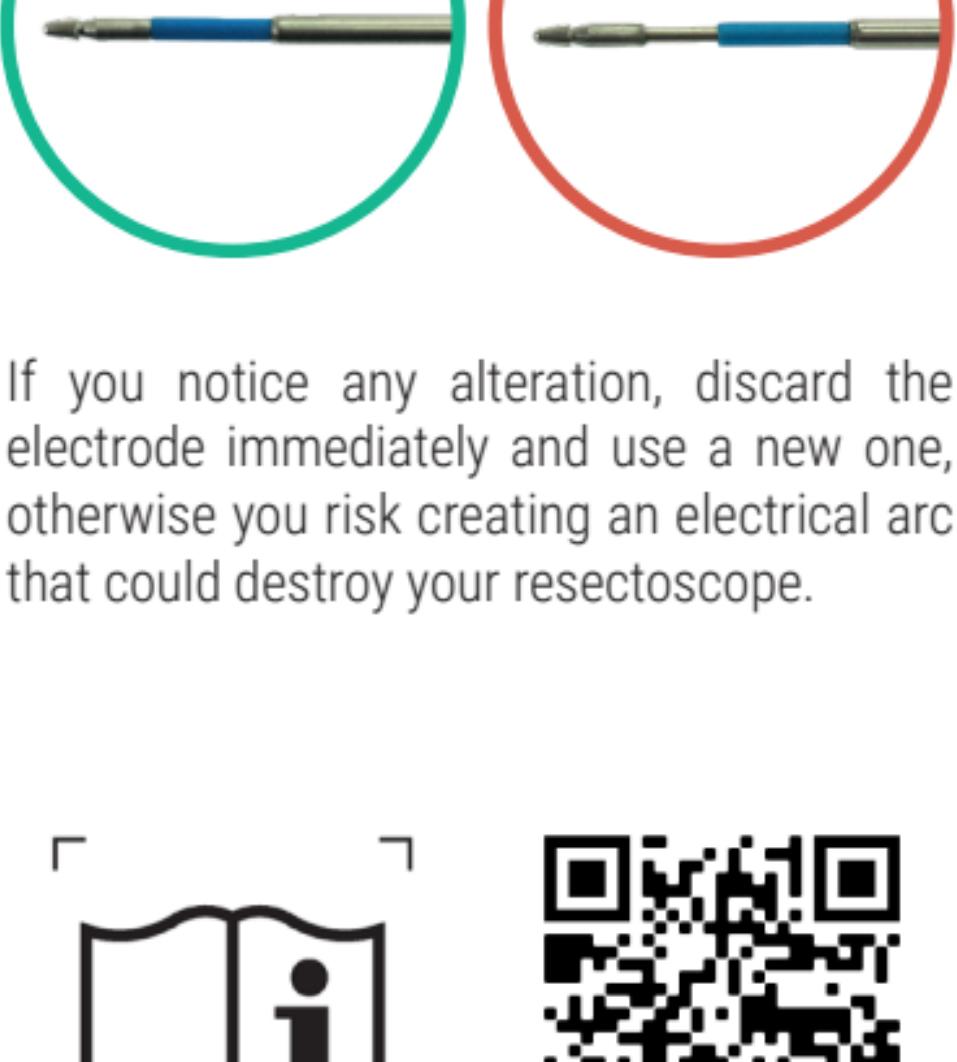

3. Alteration of the connector

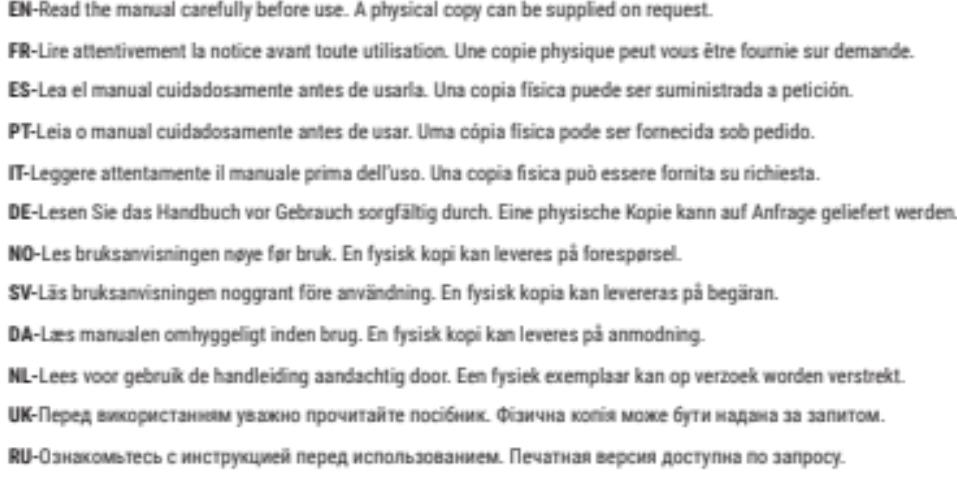If you notice any alteration, discard the electrode immediately and use a new one, otherwise you risk creating an electrical arc that could destroy your resectoscope.

www.delmont-imaging.com/downloads

EN-Read the manual carefully before use. A physical copy can be supplied on request.

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
REF


D400 100 001 ; D300 130 002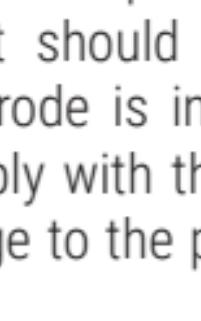

During a hysteroscopic resectoscopy, the electric current should only be activated while the electrode is in the field of view. Failure to comply with this instruction may result in damage to the product or injury to the patient.

It means that the cutting must stop as soon as the electrode is no longer visible.

In order to comply with this instruction, a stopper is now systematically inserted in the handle of the resectoscope (see attached photo).
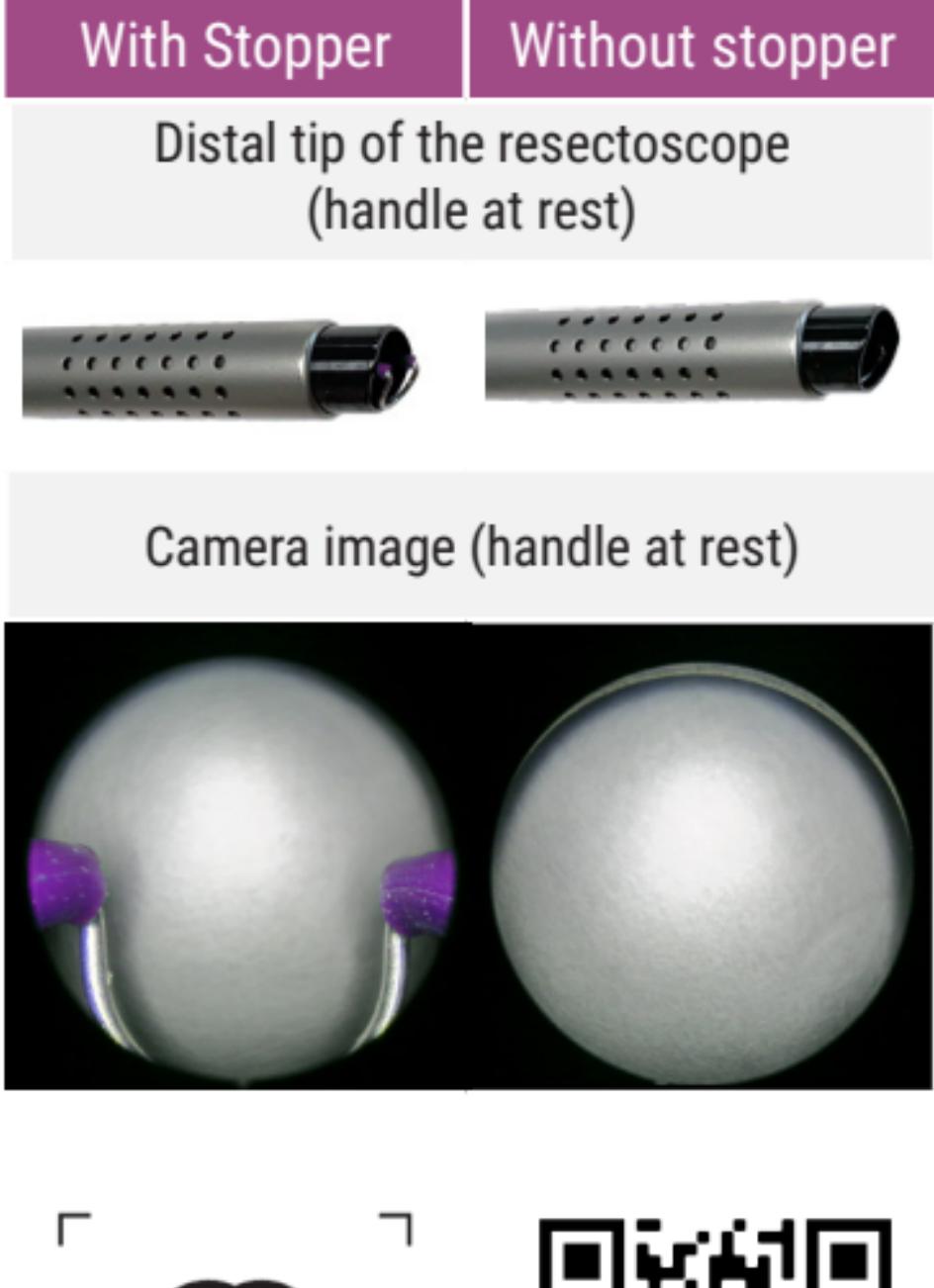


The table below shows the visual differences when the resectoscope is equipped or not with the stopper:


With Stopper

Without stopper

**Distal tip of the resectoscope
(handle at rest)**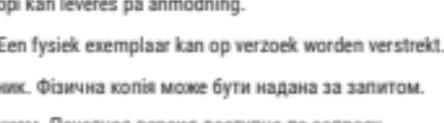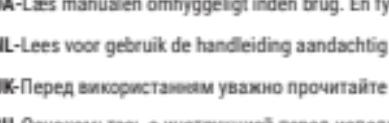

Camera image (handle at rest)

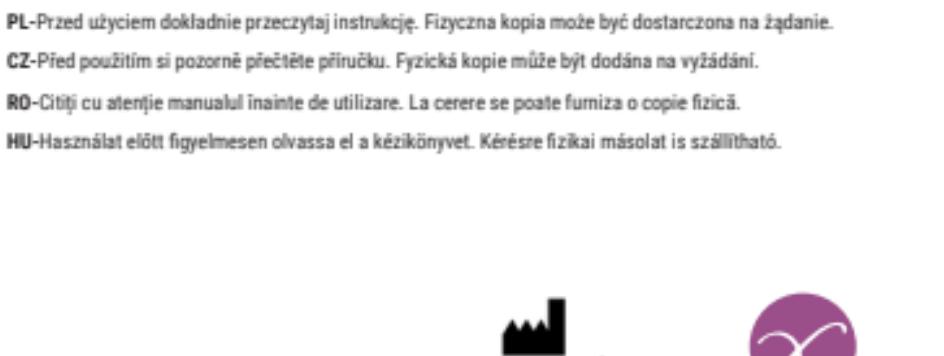D900 700 055 C EN

2020-03

2/2

CE

1304

Delmont Imaging

390 Avenue du Mistral

13600 La Ciotat - FRANCE

Tel : +33 (0) 951 513 030

Fax : +33 (0) 957 513 100

contact@delmont-imaging.com

www.delmont-imaging.com

delmont

imaging