







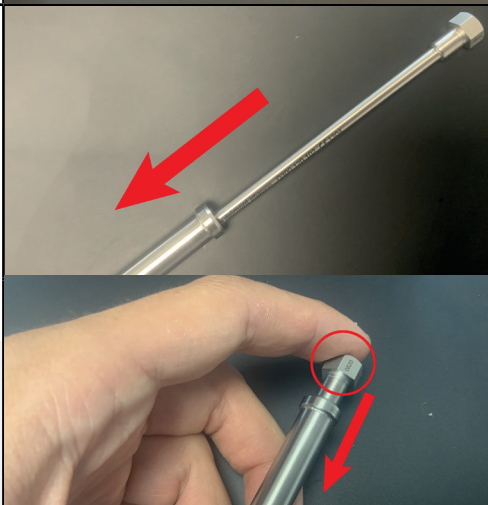

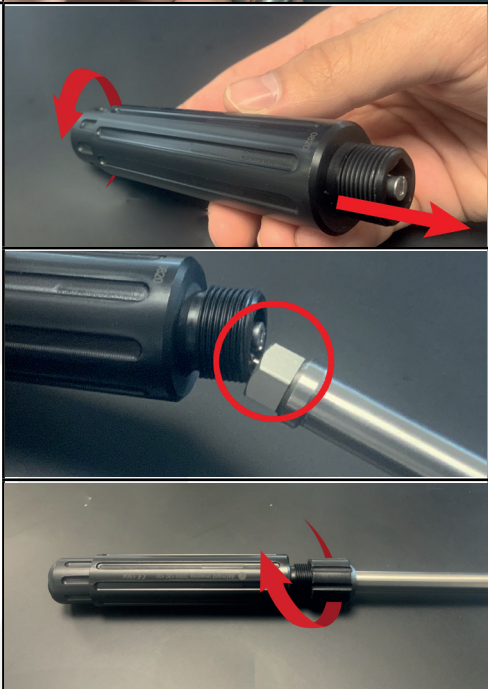


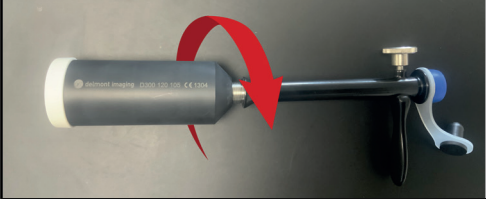

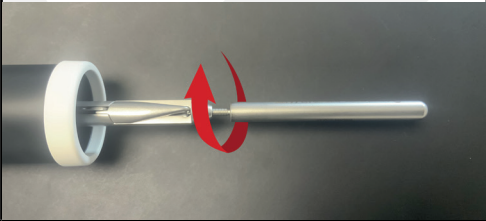


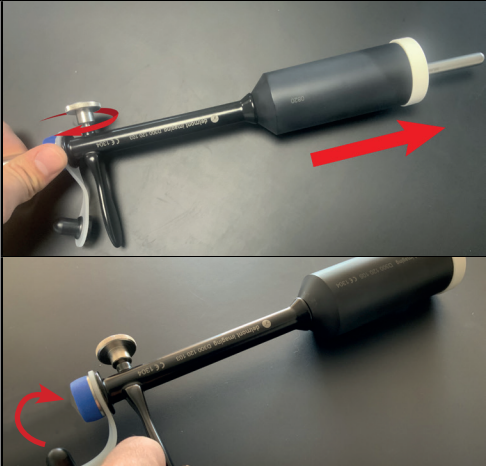
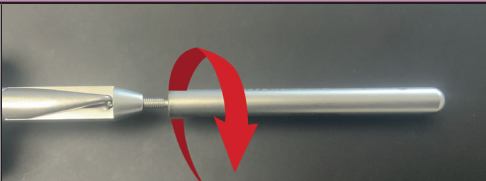
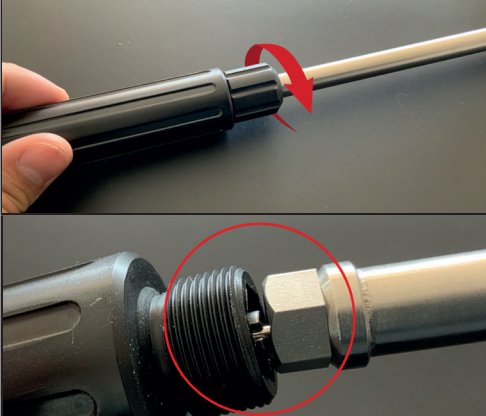
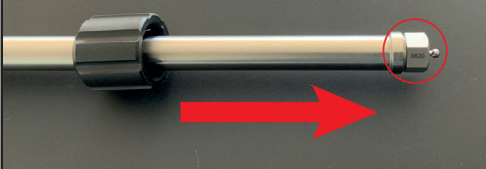



Mangeshikar Quick-start

Set elements	
	Handle with rotation module
	Insert with Pozzi forceps
	Inner sheath
	Outer sheath
	Sliding sheath
	3 Caps (Ø 35/40/45mm)
	7 Intra-uterine inserts 4x40/5x50/6x60/7x70/8x80/8x120/8x150mm

Step	Photo	Description
Assembly		
1		<p>Slide the insert with Pozzi forceps into the outer sheath.</p> <ul style="list-style-type: none"> Screw in the insert.
2		<p>On the other side of the outer sheath, insert the inner sheath.</p> <p> The insert with Pozzi forceps previously assembled to the outer sheath must fit into the inner sheath during its insertion.</p> <ul style="list-style-type: none"> Push the nut of the inner sheath against the outer sheath. You should hear a locking «click».
3		<p>Insert the assembly onto the handle.</p> <ul style="list-style-type: none"> Prepare the handle, by turning the rotation module at its base clockwise, to pull out the half-slotted screw. Turn until the module is blocked. Place the ball at the end of the sheaths/insert assembly into the half-slotted screw of the handle. <p> Ensure that the Pozzi forceps are closed tightly so that the ball comes out of the outer sheath.</p> <ul style="list-style-type: none"> Finalise the assembly by screwing the black module of the outer sheath to the handle while holding the ball in the half-slotted screw.

Step	Photo	Description
4		Assemble the sliding sheath. <ul style="list-style-type: none"> Loosen the screw to allow it to be inserted onto the rest of the assembly and open the sealing plug.
		<ul style="list-style-type: none"> Select a cap of the appropriate diameter for the patient's uterus and screw it onto the sliding sheath.
		<ul style="list-style-type: none"> Close the Pozzi forceps by turning the handle rotation module counter-clockwise and slide the sliding sheath onto the sheaths/insert assembly.
5		Assemble the intra-uterine insert. <ul style="list-style-type: none"> Select an intra-uterine insert that fits the patient's uterus and screw it onto the distal end of the Pozzi forceps insert.
Use		
1		<p>Opening the Pozzi forceps: Rotate the handle's module clockwise.</p> <p>Closing the Pozzi forceps: Rotate the handle's module anticlockwise.</p>
2		<p>Locking the Pozzi forceps position: Push the rotation module (by turning it slightly) into the handle.</p> <p>Unlocking the Pozzi forceps position: Press the rotation module (by turning it slightly) to release it from the handle.</p>

Step	Photo	Description
3		During the surgery <ul style="list-style-type: none"> After removal of the uterus, close the Pozzi forceps. Loosen the screw of the sliding sheath and remove it from the sheaths/insert assembly.
		<ul style="list-style-type: none"> Close the plug of the sliding sheath. Replace the sliding sheath in the vagina in order to maintain the seal during suturing.
Disassembly		
1		Unscrew the intra-uterine insert.
2		Remove the handle. <ul style="list-style-type: none"> Turn the rotation module as much as possible clockwise. Unscrew the module of the outer sheath.
		<ul style="list-style-type: none"> Push the ball out of the half-slotted screw with your finger.
3		Remove the inner sheath. <ul style="list-style-type: none"> Pull the inner sheath out holding it by the nut.
4		Unscrew the insert with Pozzi forceps.