




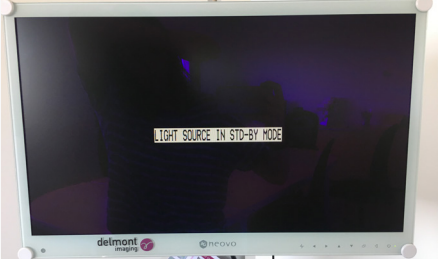

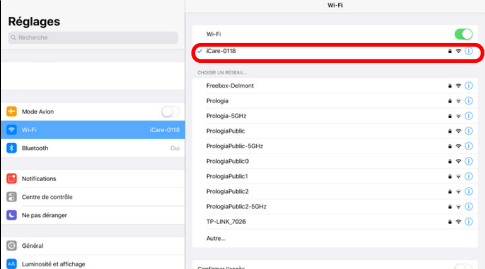
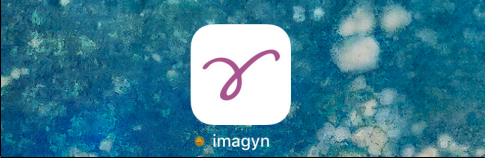
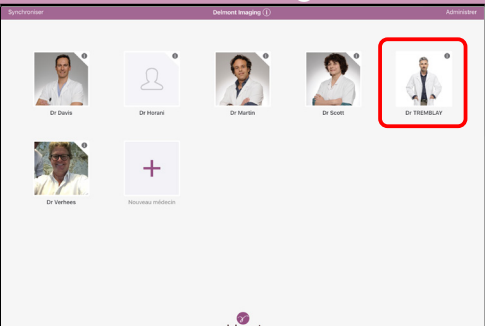
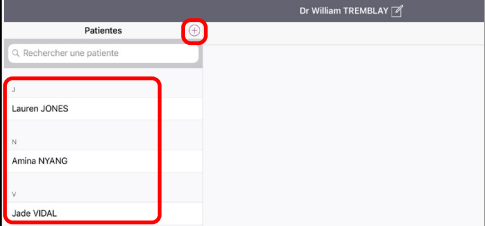
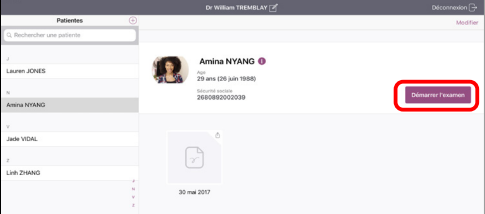
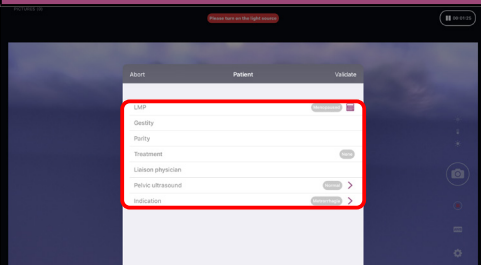











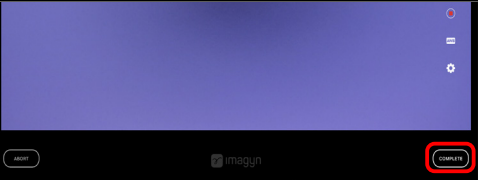
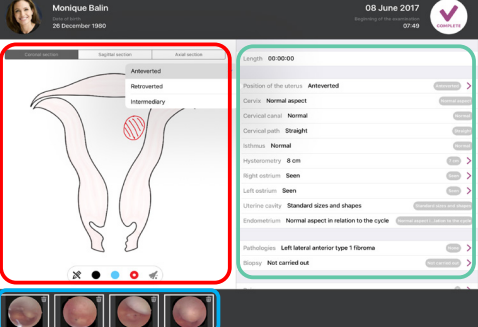
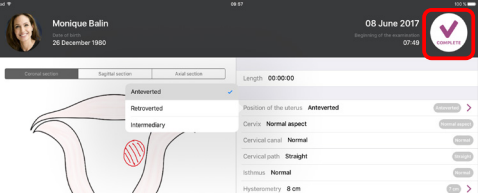
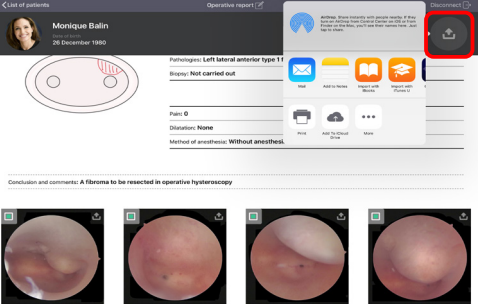


iCare+ Quick-start

Step	Photo	Description
Starting up the iCare+ column		
1		Switch on the column. <ul style="list-style-type: none"> Put the cart switch on «I» position. The green switch lights.
2		Switch on iCare. <ul style="list-style-type: none"> Put the rear switch on «I» position.
3		Wait 30 seconds for iCare to start. <ul style="list-style-type: none"> The logo flashes rapidly during the booting phase. When this phase is done, the logo flashes slowly.
4		Connect the camera head and the light cable. <ul style="list-style-type: none"> The camera on the right and the light cable on the left. <p> For the camera, align the red dots before the connection.</p>
5		Check if the monitor has turned on automatically. <ul style="list-style-type: none"> The message «Light source in Std-by mode» must appear on the screen.
6		Switch on the iPad. <ul style="list-style-type: none"> Press the on/standby button on the side.

Step	Photo	Description
7		Check the iPad connection to the iCare Wifi. <ul style="list-style-type: none"> Go on the Wifi settings.
8		Launch imagyn.
Performing an exam and the report with iCare+		
1		Selection of the account by the doctor. <ul style="list-style-type: none"> A private 6-digit code is needed to access the account.
2		Creation of a new patient. (or selects the patient in the worklist if already existing).
3		Start the exam. <ul style="list-style-type: none"> Press the corresponding button.

Step	Photo	Description
4		Fill the preliminaries fields if desired. <ul style="list-style-type: none"> No mandatory field.
5		Check the stream of the camera on the iPad. <ul style="list-style-type: none"> The image needs to be the same as on the monitor.
6		Activate the light source. <ul style="list-style-type: none"> Press on the button on the right side face of the iCare main unit.
7		Carry out a white balance. <ul style="list-style-type: none"> Film a white surface. Press the AWB button until «AWB in progress...» appears. <p>(With the camera head or press the  button on the iPad)</p> <ul style="list-style-type: none"> Keep on filming a white surface until «AWB OK» appears.
8		If necessary, adjust the settings of the camera. <ul style="list-style-type: none"> Press on the  button to access the settings panels.

Step	Photo	Description
9		Perform the hysteroscopy. <ul style="list-style-type: none"> During the exam, if necessary, you can : <ul style="list-style-type: none"> Take pictures: with the camera head or the  button on the iPad. Make videos : with the camera head or the  button on the iPad. Switch the light intensity between MIN and MAX: with the camera head or the  on the iPad. Do a numerical zoom: with the  button on the iPad.
10		Complete the exam. <ul style="list-style-type: none"> Press the corresponding button in order to go on the report edition.
11		Edit the report. <ul style="list-style-type: none"> Three different parts are available: <ul style="list-style-type: none"> Red one: Drawing on three sections of the uterus with the Apple pencil. Blue one: Drawing on the pictures taken and tag them. Green one: Filling in the fields of the report.
12		Complete the edition of the report. <ul style="list-style-type: none"> Press the corresponding button.
13		If necessary, export the report (or only the pictures). <ul style="list-style-type: none"> You can print it or send it by email for example.