
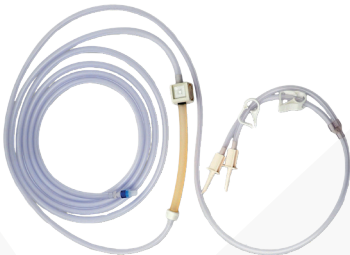

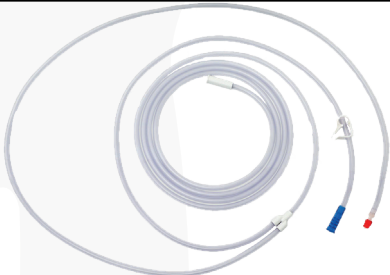


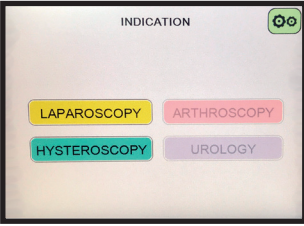
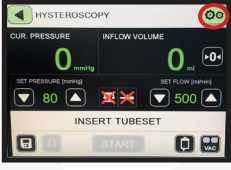

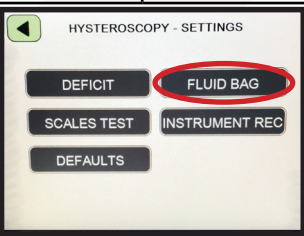
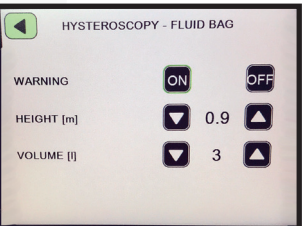
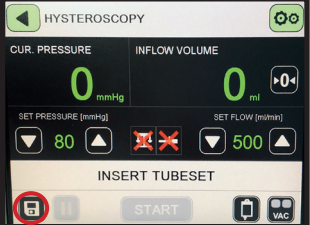


iFlow Quick-start

Set element	
	Irrigation and suction pump (Hysteroscopy programme)
Tubes	
	Single-use sterile irrigation tube Box of 10 tubes.
	Multi-use irrigation tube One tube reusable 20 times.
	Single-use sterile suction tube With 2 connectors Box of 10 tubes.
	Vacuum suction tube with filter One tube usable for a period of 30 days.




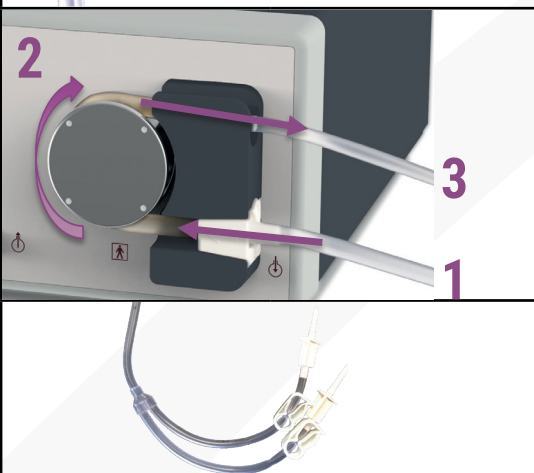






Step	Photo	Description
Set up		
1		Commissioning <ul style="list-style-type: none"> Place the pump at patient height. Switch on the pump by pressing the button on the front panel. A loading screen appears.
2		Program selection <ul style="list-style-type: none"> Select the Hysteroscopy program (highlighted).
3	 	Settings <ul style="list-style-type: none"> Select the settings button, then «Hysteroscopy».
4		<ul style="list-style-type: none"> Select the setting «Fluid bag».
		<ul style="list-style-type: none"> Turn on or off the alert when the fluid bag is almost empty. Adjust the height at which the fluid bag is positioned (height from the patient, not the floor). Adjust the volume of the fluid bag used.
5		Pressure and flow <ul style="list-style-type: none"> Choose the starting pressure and flow (recommended values: 80mmHg and 500mL/min). Save them if necessary to make them the default settings for the Hysteroscopy program.

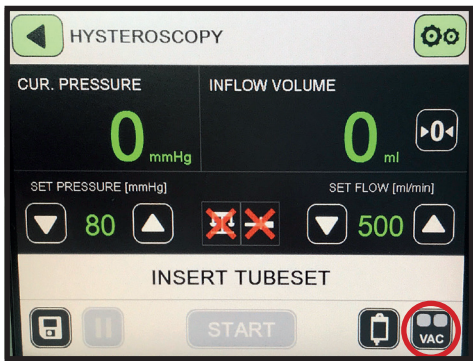
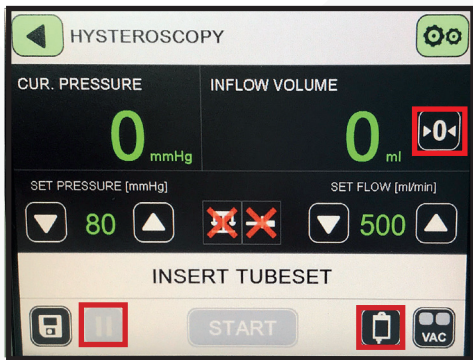








Non-sterile hands



Sterile hands

Step	Action by	Photo	Description
Connections for irrigation			
1			<ul style="list-style-type: none"> Connect the irrigation tube to the corresponding stopcock on the surgical instrument.
2			<ul style="list-style-type: none"> Retrieve the irrigation tube and insert it around the pump roller. Connect the two (or one) strikers to fluid bags.
Connections for suction			
1			<ul style="list-style-type: none"> Connect the suction tube to the corresponding stopcock on the surgical instrument.
2			<ul style="list-style-type: none"> Connect the other end of the tube to the waste collection container.
3			<p>Vacuuming</p> <ul style="list-style-type: none"> Connect the vacuum suction tube to the pump, and to the waste collection container (if present).

Step	Photo	Description
Operation		
1		Starting the pump <ul style="list-style-type: none"> Press START after insertion of the irrigation tube.
		<ul style="list-style-type: none"> Start the suction on the pump using the suction button (if necessary). Press again to switch to the maximum suction power level.
2		During the surgery <ul style="list-style-type: none"> Use the arrows to change the set pressure and flow if required. 
		<ul style="list-style-type: none"> If needed, stop the pump momentarily by pressing pause. 
		<ul style="list-style-type: none"> Reset the input volume. 
		<ul style="list-style-type: none"> When the fluid bag is almost empty, an audible alert and a message are present. Change the fluid bag and make a long press on the corresponding button which will then turn green. 
3		Stopping the pump <ul style="list-style-type: none"> Turn off the suction (if present).
		<ul style="list-style-type: none"> Press STOP to stop the pump permanently.