





















Company										
Product name	Mangeshikar	Keckstein	Clermont -ferrand	Hohl	Tintara	Donnez	SecuFix	RUMI II	Advincula Arch	Hystero-swing
Reusable										
Single use part	/	/	/	/	/	/	Transillumination cap and vacuum tube	Caps and intra-uterine inserts		/
Number of assembly steps	6	7	7	5	4	4	5	3	7	
Available caps	3	4	3 Blades	4	No cap	3	1 «Multi-purpose»	4	3	
Available intra-uterine inserts	7	3 spiral inserts 6 working inserts	7	3 spiral inserts 9 working inserts	7 working inserts	3	1 (adjustable lenght)	6	5	
Uterus fixation system	Pozzi forceps	Screw with spiral insert	Cervical screw	3 spiral inserts	Pozzi forceps	Insert	Vacuum	Balloon	Cervical screw	
System for vaiginal sealing	Vaginal cap	Silicone seals		Vaginal cap	Insert body	Vaginal cap	Vaginal cap	Intrauterine balloon	Manipulator body	
Sealing after removal of the uterus										

Company										
Product name	Mangeshikar	Keckstein	Clermont -ferrand	Hohl	Tintara	Donnez	SecuFix	RUMI II	Advincula Arch	Hystero-swing
Blue test cannula	✗	✗	✓	✗			✗	✓		✗
Handle with mecanical rotation of the intra-uterine insert	✗	✓		✗		✓	✗	✓		✗
Overall manipulator handling (Ante + Retro)	Good	anteversion 95°, retroversion 30°, maintains continuous blocked angulation	Medium (5 stop positions from 0 to 90°)	Good	Good	Medium	Good	90° anteversion, 50° retroversion, maintains blocked angulation	Good (Manual anteversion and retroversion)	Good
Quality of the highlighting of the cutting area	Very good (white ceramic ring)		Medium (blade and no cap)	Very good (white ceramic ring)	Bad (no cap)	Very good (white ceramic ring)	Very good (transillumina-tor)	Good		Medium (blade and no cap)
Resistance to cutting	Good (white ceramic ring)		NA. Blade and no cap	Good (white ceramic ring)	Good	Good (white ceramic ring)	Medium (plastic)	Medium (plastic)		NA. Blade and no cap


Mangeshikar



SecuFix



Keckstein

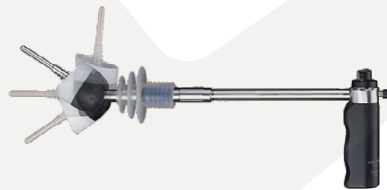

Hohl




Donnez



Tintara



Clermont -ferrand

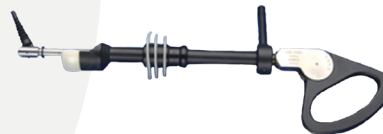

Hystero-swing



RUMI II



Advincula Arch

