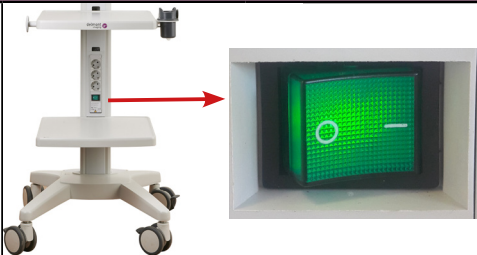

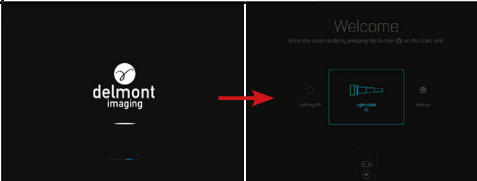


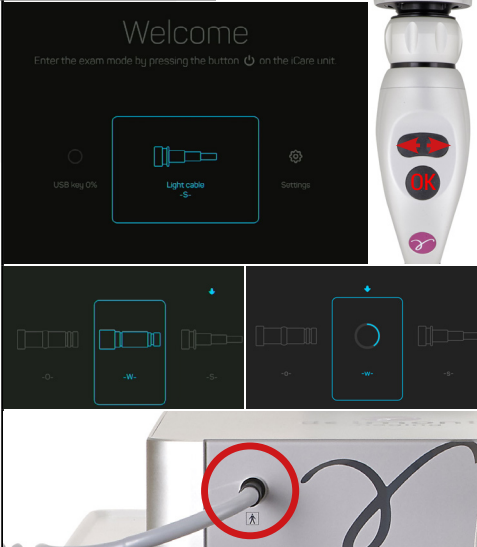


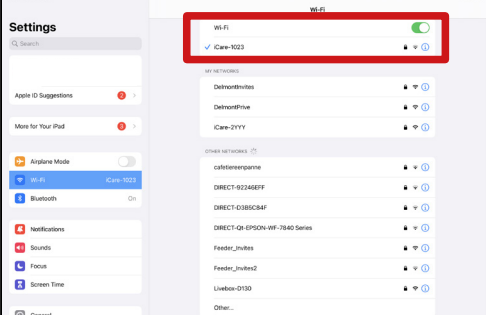
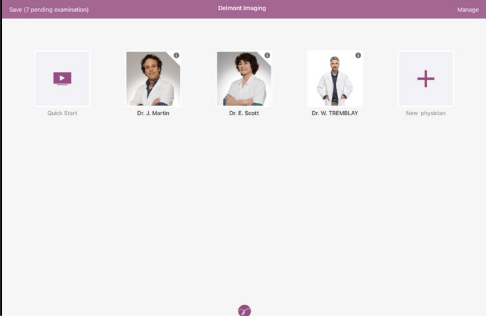


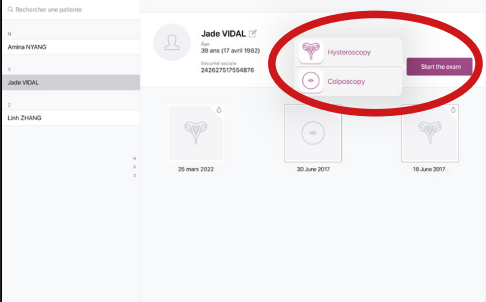
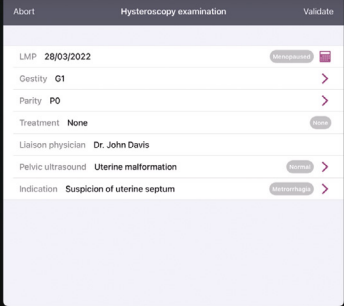



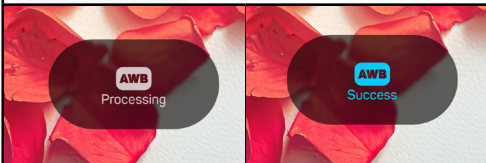




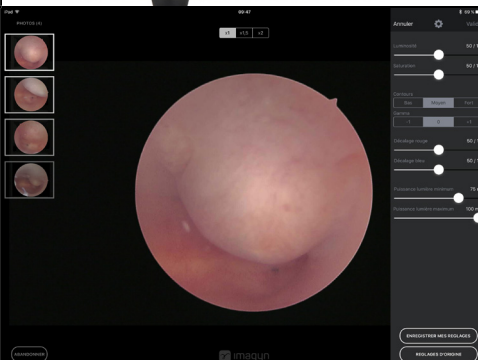

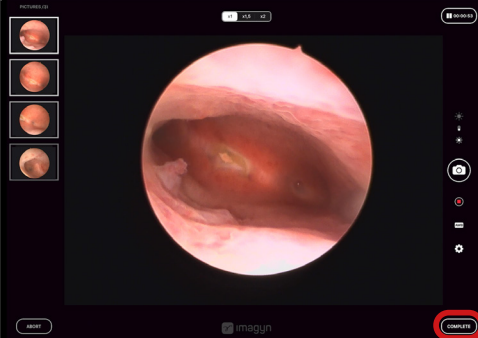


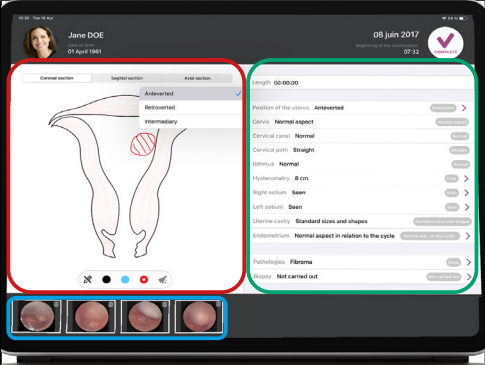
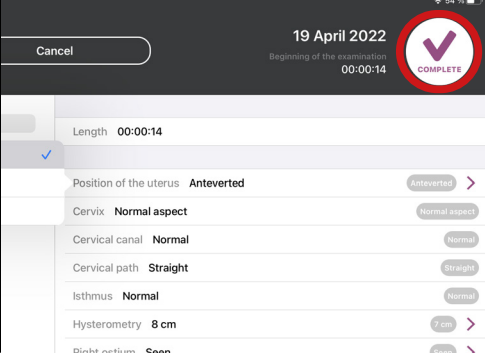
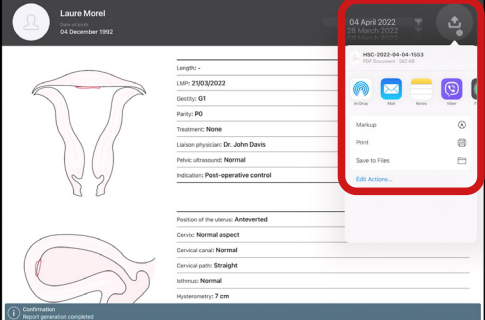
iCare+ Quick-Start

Step	Photo	Description
Starting up the iCare+ tower		
1		Switch on the cart. <ul style="list-style-type: none"> Put the cart switch on «I» position. The green switch lights.
2		Switch on iCare. <ul style="list-style-type: none"> Put the switch on the rear panel of the control unit on «I» position.
3		Wait 30 seconds for iCare to start. <ul style="list-style-type: none"> During the powering, a boot screen appears. When this phase is done, the home screen appears.
4		Connect the camera head on the right connector on the front panel of the control unit. <p> Align the dots before plugging the camera head.</p>
5		Connect the light cable on the left connector on the front panel of the control unit. <p>If SLIDE iCare:</p> <ul style="list-style-type: none"> On the home screen, select the «Light cable» menu thanks to the camera head buttons. Choose the suitable standard of the light cable you want to use. Confirm then connect your light cable to iCare. <p> If non SLIDE iCare: only Storz standard.</p>

Step	Photo	Description
imagyn application setup on the iPad		
1		Unlock the iPad. <ul style="list-style-type: none"> Press the standby button on the side of the iPad. Swipe up from the bottom of the screen to display the home screen.
2		Verify that the iPad is connected to the iCare Wi-Fi network. <ul style="list-style-type: none"> Select the Wi-Fi tab in the Settings application of the iPad.
3		Launch imagyn  and select the doctor's account. <ul style="list-style-type: none"> A private 6-digit code is required to access the account.
4		Creation of a new patient. <ul style="list-style-type: none"> Press the ⊕ button to create the patient file. If the patient is already in the database, select her file from the list. A search bar is available.
5		Start the exam. <ul style="list-style-type: none"> Press the corresponding button (Hysteroscopy or Colposcopy).

Step	Photo	Description
Settings before the exam		
1		<p>Fill the preliminaries fields if desired.</p> <ul style="list-style-type: none"> Once the filling done, select « Validate ». No fields are mandatory. <p>A selection of pre-written answers is available by clicking on the purple arrows.</p> <p>The grey boxes allow the automatic filling of the field.</p>
2		<p>Switch on the light source.</p> <ul style="list-style-type: none"> Press the standby button on the right side face of the control unit. On the monitor, the home screen is replaced by the camera image that is displayed.
3		<p>Check the camera image on the iPad.</p> <ul style="list-style-type: none"> The image should be the same as the one on the monitor.
4	<p>Long press to launch the white balance</p>  	<p>Carry out a white balance.</p> <ul style="list-style-type: none"> Film a white surface. Press the button until «AWB in progress...» appears. <p>(With the camera head or press the AWB button on the iPad)</p> <ul style="list-style-type: none"> Keep on filming a white surface until «AWB Success» appears.

Step	Photo	Description
Exam realization and report edition		
1	 <p>Short or long press</p> <ul style="list-style-type: none"> - Anti-moiré - Gain - Zoom - No action <p>- Short press to take pictures.</p> <p>- Long press to start/stop video recording.</p>	<p>Perform the hysteroscopy.</p> <ul style="list-style-type: none"> • During the exam, if necessary, you can: <ul style="list-style-type: none"> - Take pictures. (With the camera head or the  button on the iPad) - Make videos. (With the camera head or the  button on the iPad) - Other actions: anti-moiré, gain, zoom, no action. (With the programmed camera head button into the «user actions»  of iCare's settings.) - Adjust the light intensity. (With the «+» and «-» buttons on the right side face of the control unit).
2		<p>If necessary, adjust the settings of the camera from the iPad.</p> <ul style="list-style-type: none"> • Press on the  button to access the settings panel.
3		<p>Complete the exam.</p> <ul style="list-style-type: none"> • Press the corresponding button in order to go on the report edition.

Step	Photo	Description
4		Edit the report. <ul style="list-style-type: none"> Three different parts are available: <ul style="list-style-type: none"> - Red: Drawing on three sections of the uterus with the Apple stylus. - Blue: Drawing on the pictures taken and tag them. - Green: Filling in the fields of the report.
5		Complete the edition of the report. <ul style="list-style-type: none"> Press the corresponding button.
6		If necessary, export the report. <ul style="list-style-type: none"> Many export possibilities are available: sending by secure e-mail address, printing or sharing in a shared network file.