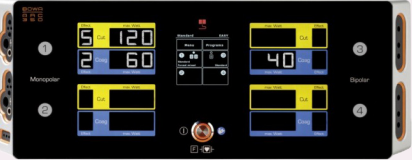


Generator	Compatible plug	Cable and adapter to use	Program verification	Recommended settings	Footswitch
Erbe Vio 300 D 	MF plug 	 Reference: D200 190 001	The bipolar resection program is not automatically displayed: select it from the list of available programs. If not, it has not yet been created. Please contact the care center staff to create it.	Cut: Effect 3 Coag : Effect 3  The currents to be used are Bipolar CUT ++ and Bipolar SOFT COAG ++ .	
	MF-2 plug 	 Adapter reference: D200 190 033  Cable reference: D200 190 001			
Erbe Vio 3 	MF plug 	 Reference: D200 190 001	All programs are pre-installed. When you connect a cable, the generator automatically recognizes it and displays the appropriate program.	Cut: Effect 5 Coag : Effect 5	
	MF-U plug 	 Reference: D200 190 034 or			
		 Ref. : D200 190 033  Ref. : D200 190 001			

Generator	Compatible plug	Cable and adapter to use	Program verification	Recommended settings	Footswitch
Valleylab ForceTriad 	LigaSure 2 plug  It is positioned at the bottom, next to the orange pedal logo.	 Reference: D200 190 000	The bipolar resection program is optional and must have been purchased and activated. The Covidien part number for the bipolar resection option is FTBRKIT.  The generator is compatible if the software version is at least 3.30. In theory, if the program has been purchased and installed, a sticker stating «Bipolar resection» has been affixed to the front of the generator below the corresponding socket.	Cut: Effect 3 Coag: Effect 3	 The connector for this specific pedal is orange, and the Covidien part number for this pedal is FT6009.
Valleylab ForceFT10 	LigaSure/ Bipolar plug 	Reference: D200 190 000	On this generator, the bipolar resection program is automatically present.	Cut: Effect 4 Coag: Effect 4	
Storz Autocon II 400 	MF plug 	 Reference: D200 190 001	The bipolar resection program to choose is «Saline-bi-TUR-P».  This program is only available on «Bipolar» or «High end» models.	Cut: Effect 5 Coag: Effect 5  The currents to be used are: Saline-C-Cut Saline-Coag	

Generator compatibility with ReseCare and MaxiCare

Generator	Compatible plug	Cable and adapter to use	Program verification	Recommended settings	Footswitch
Storz Autocon III 400 	Bipolar plug 	 Reference : D200 190 008	When you connect a cable, the generator automatically recognizes it and displays the appropriate program («Bip. resection» for bipolar resection).  This recognition system is not available in the USA and is deactivated for the UH 400U version.	Cut: Effect 2 Coag: Effect 3	
KLS Martin Maxium 	Bipolar B1 or B2 plug 	 Reference : D200 190 006	Check that the bipolar resection program has been installed. This is an option for Reference 80-093-00-04.  The generator is compatible only if the software version is at least 6.	Cut: G3 Coag: 80 Watt Bipol pure cut current	
Delmont Imaging iSpark et iSpark+ 	«out 1» or «out 2» plug 	 Reference : D200 190 007	On iSpark+, check that the «Hystero bi-cut» and «Hystero bi-coag» programmes have been installed. When you connect the cable, check that these 2 programmes are selected automatically, otherwise select them manually.	Cut: Effect 3 Coag : Effect 3	

Generator	Compatible plug	Cable and adapter to use	Program verification	Recommended settings	Footswitch
Bowa ARC 350 	Bipolar 3 plug 	 Reference: D200 190 008	Check that the bipolar resection program has been installed. This is an option for reference: BOWA 900-395. When this program is installed, when you connect a cable, the generator automatically recognizes it and displays the appropriate program.	Cut: Effect 2 Coag: Effect 2	
Bowa ARC 400 	Bipolar 3/ 4 plug 	Reference: D200 190 008	When this program is installed, when you connect a cable, the generator automatically recognizes it and displays the appropriate program.		
Gynecare/ Olympus Versapoint II 		 Ref.: D200 190 008  Ref.: D200 186 006	The bipolar resection program is automatically present. When you connect the adapter, the generator recognizes it and displays the appropriate program.	Cut: 120 Coag: 90	