


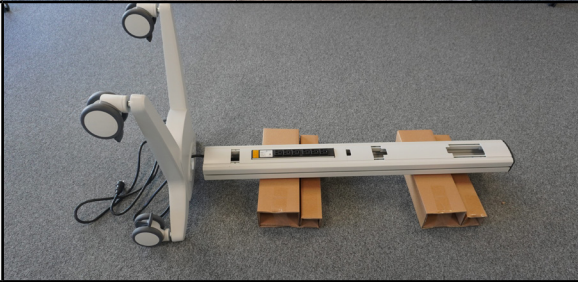


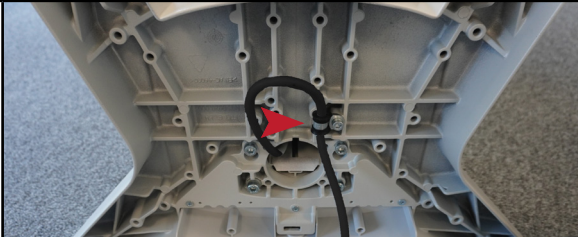

Assembly instructions: iCare+ cart



Assembly time: 30 min to 1 hour



Materials required: 3 Allen keys (supplied)
All parts to be assembled are supplied complete with screws

Step	Photo	Description
Column and stand assembly		
1		<ul style="list-style-type: none"> Unpack all parts from the box.
2		<ul style="list-style-type: none"> Position the stand vertically on the 2 rear wheels. Position the column horizontally and elevate it using the 2 small cardboard wedges from the packaging. Align the column with the dedicated space on the stand.
3		<ul style="list-style-type: none"> Take the electric cable from the column and pass it through the hole in the base. Insert the 4 screws to assemble the column and base. Tighten with the Allen key. <p> <i>There is only one screw out of 4 with a nut, and its location is of no importance.</i></p>
4		<ul style="list-style-type: none"> Unscrew the cable gland. Then pass the cable through it. Screw the cable gland back on. <p> <i>Remember to route the cable towards the rear of the trolley.</i></p>



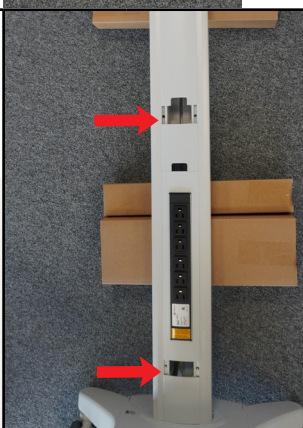







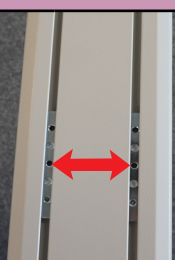
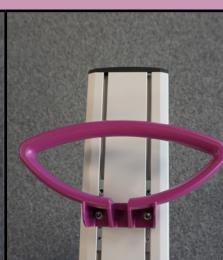
delmont imaging










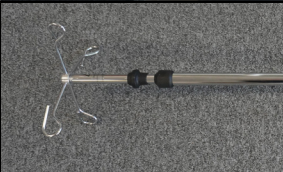




Athélia IV - 297, Av. du Mistral - 13 600 La Ciotat, FRANCE

Tel. : +33(0) 9 51 51 30 30 - fax : +33(0) 9 57 51 31 00

www.delmont-imaging.com - contact@delmont-imaging.com

SAS au capital de 252 400 euros - RCS Marseille 817 710 411 - N° de TVA Intracommunautaire : FR 81 817 710 411

Step	Photo		Description
Fixing shelves to the column			
1		Top shelf (with logo on front)	 Bottom shelf
2			<ul style="list-style-type: none">• Locate the shelf positions.• Secure them with 2 screws each.• Tighten the screws with Allen key. <p> Move the plastic covers if they block the fastener.</p>
Mounting the monitor arm on the column			
			 <ul style="list-style-type: none">• Remove the plastic cover of the monitor arm and position it on the column.• Engage the 4 screws and tighten them with the Allen key.• Put the plastic cover back on the monitor arm. <p> Move the plastic covers of the column if they block the fastener.</p>
Handle mounting			
			 <ul style="list-style-type: none">• Raise the cart back on its wheels.• At the rear of the column, attach the handle with two screws to the dedicated location (middle holes).• Tighten the screws with the Allen key.

Step	Photo		Description
Accessories assembly			
1			<ul style="list-style-type: none">On the right-hand side of the top shelf, secure the basket support with 2 screws in the left-most holes.Once mounted, place the grey covers over the screw heads.
2			<ul style="list-style-type: none">On the same side, in the remaining hole, fix the cable holder.To the left-hand side of the top shelf, assemble the second cable holder in the same way.
3			<ul style="list-style-type: none">Add the hard grey cover on both camera cable holders.
4			<ul style="list-style-type: none">Loosen the 2 screws on the monitor arm.Position the VESA monitor support as shown opposite and secure with the 2 screws.
5			<ul style="list-style-type: none">Hand-tighten hooks to medical IV pole. <p> Once the hooks have been screwed onto the medical IV pole, they cannot be removed.</p>
6			<ul style="list-style-type: none">Insert the medical IV pole into the dedicated slot at the top of the column. <p> Insert the medical IV pole pin in the right place.</p>