



# Check-list before use

# Quick-start ReseCare

- 1 2,9mm Hysteroscope
- 2 Passive or active working element
- 3 17,5Fr. Inner sheath
- 4 18,5Fr. Outer sheath
- 5 Resection electrodes
- 6 Light cable
- 7 Endoscopy camera
- 8 Resection cable
- 9 Irrigation tube
- 10 Suction tube

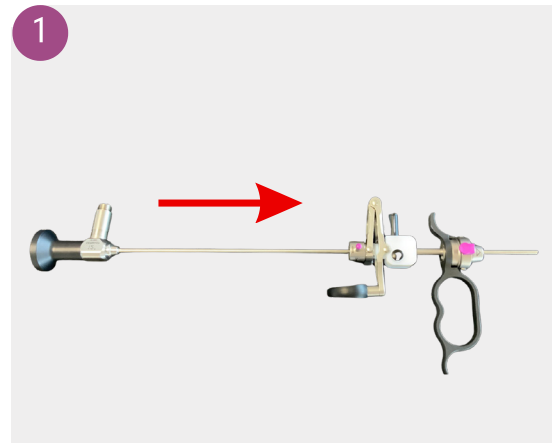


1 Insert the hysteroscope into the working element.

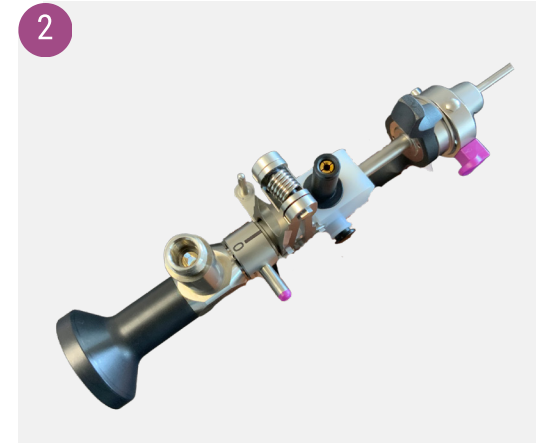


2 The « 0 » of the working element must be oriented upwards and positioned facing the light connector




3 Turn the bayonet to lock the connection.

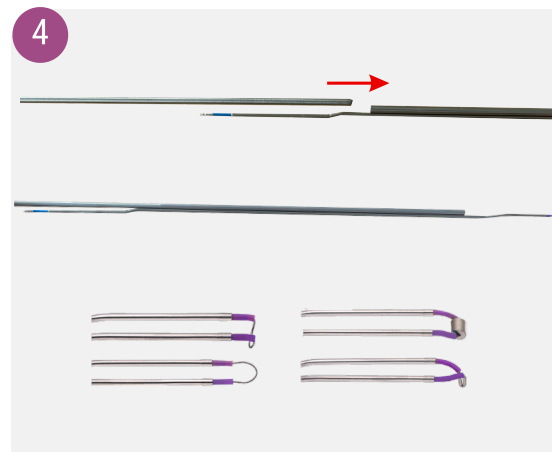


4 Choose the appropriate electrode and insert the distal part of the hysteroscope into the electrode tube.

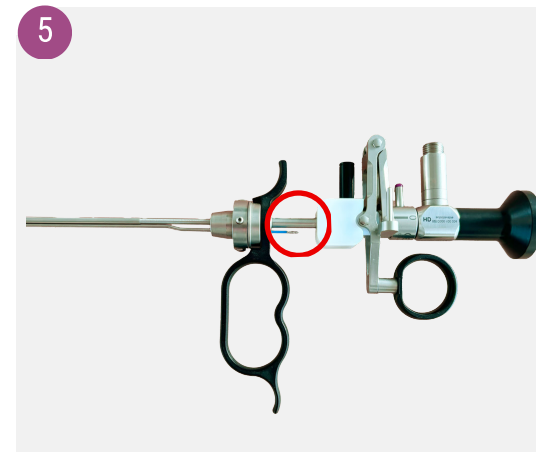


5 Insert the electrode into the dedicated hole in the working element until it reaches the white block, then push until you hear a locking click.

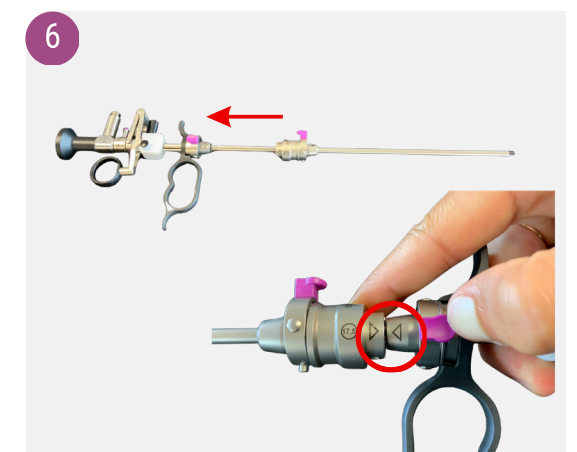


6 Next, insert the inner sheath. Align the two arrows and then push until you hear a locking click to connect the two parts.

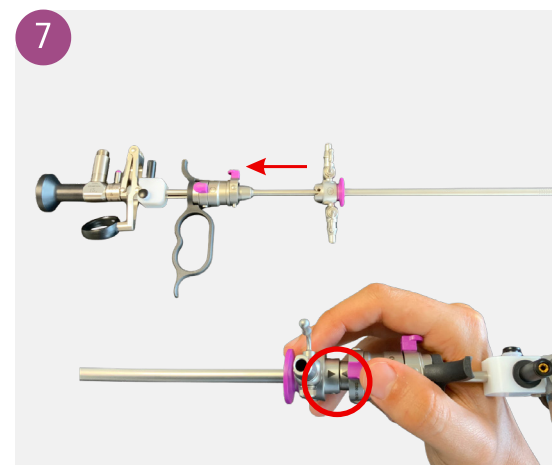

7 Finally, insert the outer sheath by aligning the two arrows and then push until you hear a locking click.

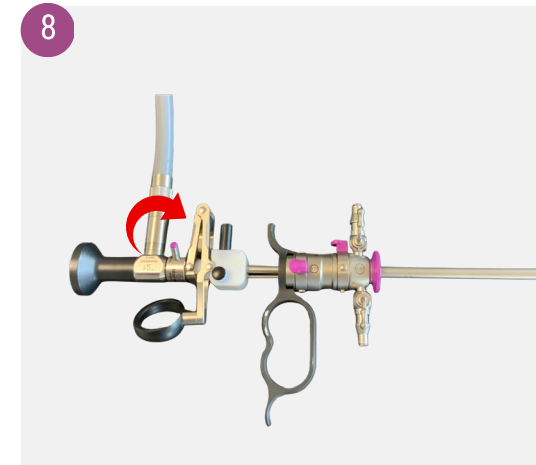


8 Connect the light cable by screwing it into the dedicated connector on the hysteroscope.

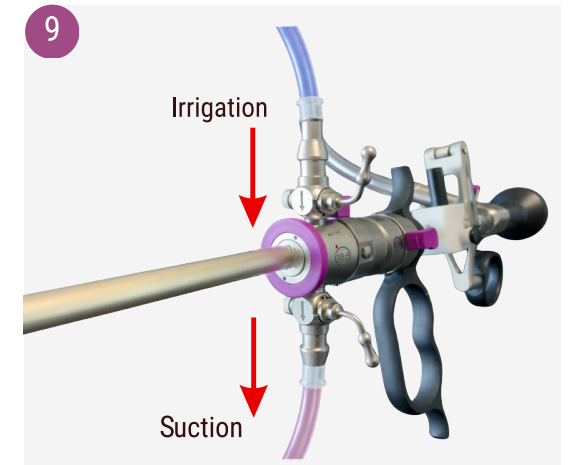


9 Connect the irrigation and suction tubes to the dedicated stopcocks on the hysteroscope.


# Quick-start ReseCare

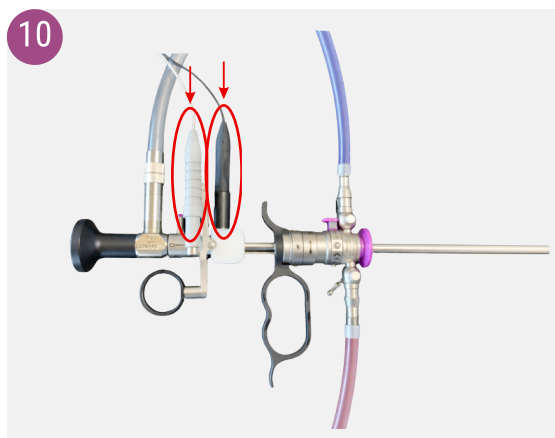


10 Connect the two ends of the bipolar cable (each connector has a different diameter).

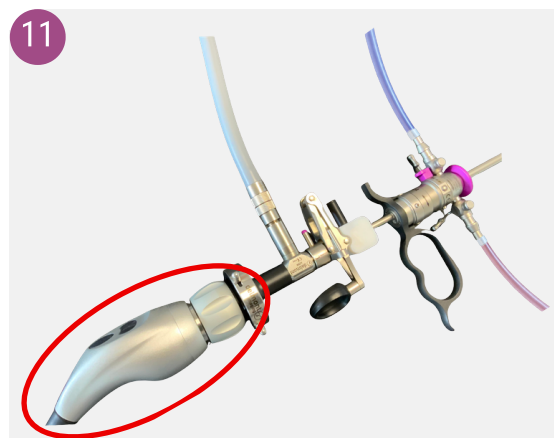

11 Finally, connect the endoscopy camera to the hysteroscope eyepiece.

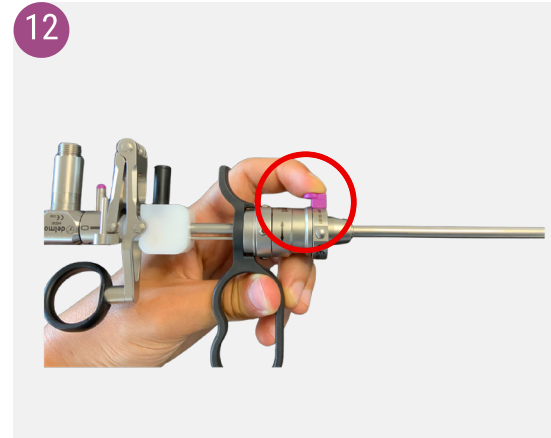

12 For disassembly, after disconnecting all ReseCare associated products, remove the outer sheath by pressing the Quick-lock button.

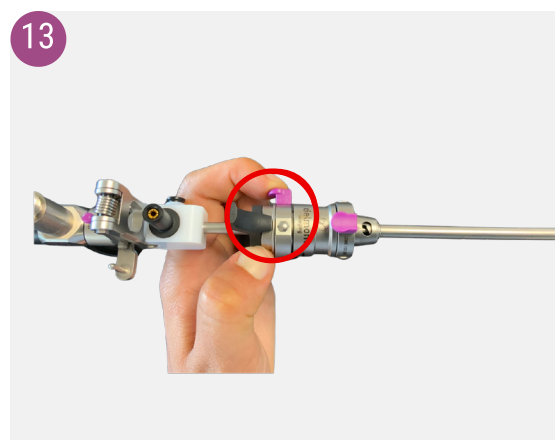

13 Disconnect the inner sheath by pressing the second Quick-lock button.



14 Remove the resection electrode by pressing the black push button on the working element.



15 Open the bayonet by turning it and remove the hysteroscope. Slide it out, keeping it straight, until it is completely removed.



Scan this QR code to watch the user guide video:



297 avenue du Mistral  
13600 La Ciotat, FRANCE



contact@delmont-imaging.com



+33 (0)9 51 51 30 30



www.delmont-imaging.com